# Over Line

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### Over Line 4 Technologies for a younger-looking skin

*Oxyenergy, the innovative system by Over Line, combines oxygen-therapy with radio-frequency, micro-dermabrasion and chromo-frequency, four different technologies to regenerate the skin of face, neck and décolleté. The device is compact and safe and is perfect to fight skin ageing and sun damages.*

*This important treatment is performed with a gentle rotating movement of a handpiece onto the skin. Micro-dermoabrasion is used to remodel the surface of the skin and consists in the mechanical abrasion of the damaged surface layers of epidermis, removal of dead cells, and stimulation of cell renewal. Oxygen-therapy is invigorating and toning for the epidermis. It stimulates cellular respiration, increases the tone, giving the skin a more elastic, compact and luminous effect after the first application. The treatment is not invasive, has no side effects and is absolutely painless. Oxygen has antibacterial and anti-inflammatory properties, and is a great ally to reduce irritation.*

*Oxygen comes out of a handpiece that beauticians must hold 1 cm far from their client's skin. Beautician's movements must be slow, start from the centre of the face towards the hairline and repeated if wrinkles are very deep. Radio Frequency is conveyed by means of a handpiece which disperses heat into deep subcutaneous tissues without damaging the epidermis. The heat produced by the diathermic action provokes an increased amount of oxygen carried to tissues, speeds up cell metabolism, elimination of catabolic wastes and stimulates the production of collagen, elastin and hyaluronic acid, for a progressive lifting effect. Beauticians can alternate Radio frequency with Lifting Treatment.*

Oxyenergy Overline Microdermoabrasione / Oxyenergy Overline Micro dermoabrasion

Oxyenergy Overline Trattamento con Ossigeno / Oxyenergy Overline oxygen-therapy

Oxyenergy Overline Trattamento Radiofrequenza / Oxyenergy Overline Radio-frequency treatment

Oxyenergy Overline Trattamento Cromo frequenza / Oxyenergy Overline Chromo-frequency treatment

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*The latter is performed by means of a specific handpiece fitted with two rotating spheres, especially designed to work even on the most difficult areas. The Chromo Frequency considers the skin imperfection as an inharmonic vibration of cells, which can be led to their correct balance using energy and so by the vibration of one or more colours. The Led Mask, combined with specific alginate masks, can give to the skin its correct balance and used with other esthetic treatments it increases the final result. As all Over Line equipments, Oxyenergy, too, is easy to use. In fact, beauticians can control every step necessary to perform the treatment correctly. Oxyenergy offers a new youth, the skin looks smoother and more nourished with an important reduction of ageing signs caused by prolonged exposure to sun rays and environmental damage. The application of cosmetics creams and serums will complete the treatment.*